




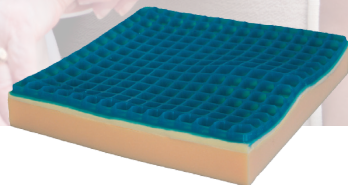

## SEATING SOLUTIONS

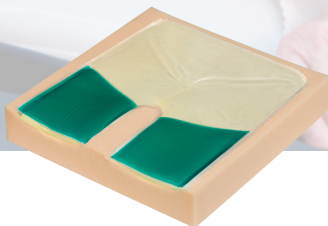

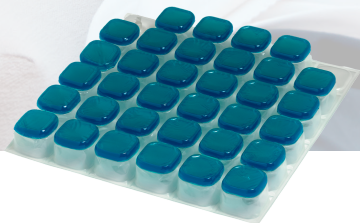

Trulife is an internationally based and managed group engaged in the creation, development, manufacture and marketing of niche healthcare products. In addition to Relax Seating Solutions the Group activities encompass Orthopaedics, Prosthetics, Breastcare and Gel positioners for use in the operating room.

- 3 A note on pressure sore prevention**
- 4 Gelcell**
- 5 Gelcell G2**
- 6 Duogel**
- 7 DuoProfile**
- 8 Easy**
- 9 Easy High Profile**
- 10 Trio**
- 11 Covers & Accessories**


## Prevention is Better than Cure

Pressure sores are areas of damaged skin and tissue that develop when prolonged periods of uninterrupted pressure cuts off the circulation to vulnerable parts of the body. Without adequate blood flow, the affected tissue dies.

People who are most vulnerable to the onset of pressure sores include those living with paralysis or using a wheelchair and any person with impaired mobility sensation.

Trulife's Relax range of Seating Solutions offer a wide variety of pressure relieving cushions for wheelchair users and people of restricted mobility who are at increased risk of developing pressure sores.

The range has been designed to redistribute weight and provide exceptional pressure relief and optimum comfort at an affordable cost, bringing a better quality of life to the user.

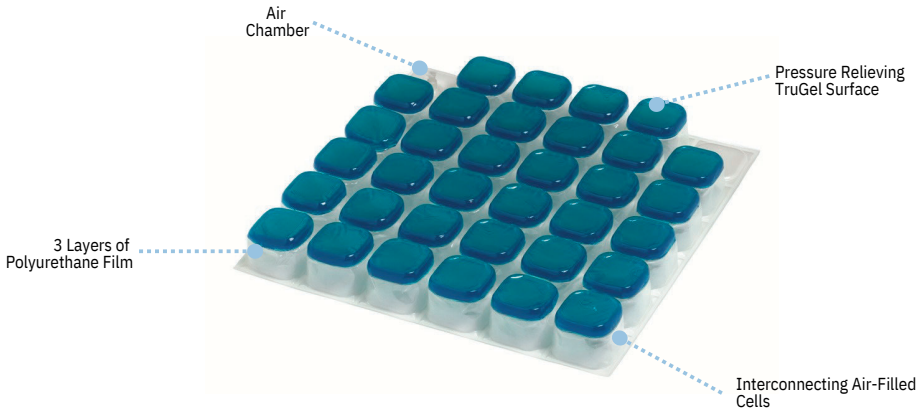

## Features & Benefits


- The unique combination of Air and TruGel offers pressure relief and comfort.
- The interconnected cell design allows movement of air through the cushion.
- The TruGel, on top of each cell, enhances pressure distribution and minimises the effects of shear.
- The Gelcell has 3 layers of polyurethane film making it highly durable.
- The cushion is easy to use and clean.

| Product Code | Dimensions* (W x L x H)              | Cushion Weight |
|--------------|--------------------------------------|----------------|
| RG004-1514   | 38cm x 36cm x 4cm / 15" x 14" x 1.5" | 1.4kg/3.1lb    |
| RG004-1616   | 40cm x 40cm x 5cm/ 16" x 16" x 2"    | 1.5kg/3.3lb    |
| RG004-1717   | 43cm x 43cm x 5cm/ 17" x 17" x 2"    | 1.6kg/3.5lb    |
| RG004-1816   | 45cm x 40cm x 5cm/ 18" x 16" x 2"    | 1.6kg/3.5lb    |
| RG004-1818   | 45cm x 45cm x 5cm/ 18" x 18" x 2"    | 1.7kg/3.7lb    |
| RG004-2018   | 50cm x 45cm x 5cm/ 20" x 18" x 2"    | 1.9kg/4.2lb    |

*The weight limit for the Gelcell cushion is 130kg/287lbs*

*\*A tolerance of +/- 10mm is applied to dimensions*

*Comes with a standard black cover and Handi pump*

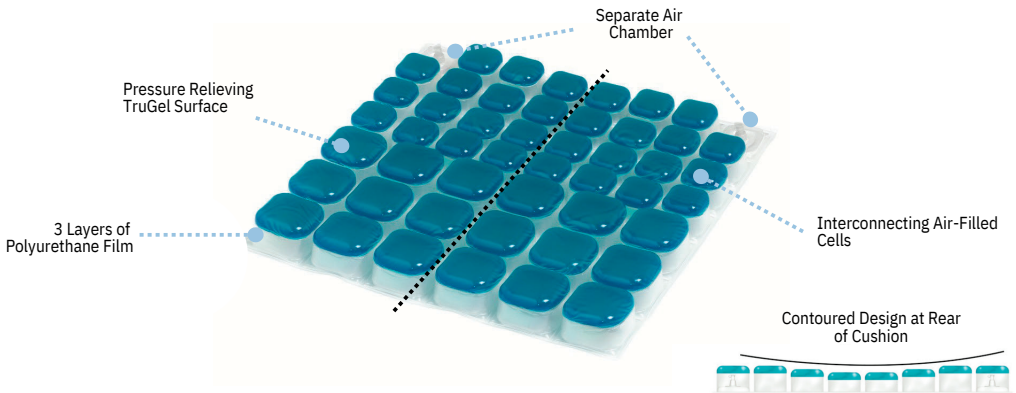

## Features & Benefits


- The unique combination of Air and TruGel offers pressure relief and comfort.
- Two separate air chambers and contoured cells provide pressure redistribution.
- The TruGel enhances pressure distribution and minimises the effects of shear.
- The Gelcell G2 has 3 layers of polyurethane film making it highly durable.
- The cushion is easy to use and clean.

| Product Code | Dimensions* (W x L x H)              | Cushion Weight |
|--------------|--------------------------------------|----------------|
| RG005-1414   | 36cm x 36cm x 4cm / 14" x 14" x 1.5" | 1.3kg/2.9lb    |
| RG005-1515   | 38cm x 38cm x 4cm / 15" x 15" x 1.5" | 1.3kg/2.9lb    |
| RG005-1616   | 40cm x 40cm x 4cm / 16" x 16" x 1.5" | 1.5kg/3.3lb    |
| RG005-1618   | 40cm x 45cm x 4cm / 16" x 18" x 1.5" | 1.6kg/3.5lb    |
| RG005-1717   | 43cm x 43cm x 4cm / 17" x 17" x 1.5" | 1.6kg/3.5lb    |
| RG005-1816   | 45cm x 40cm x 4cm / 18" x 16" x 1.5" | 1.6kg/3.5lb    |
| RG005-1818   | 45cm x 45cm x 4cm / 18" x 18" x 1.5" | 1.6kg/3.5lb    |

*The weight limit for the Gelcell G2 cushion is 130kg/287lbs*

*\*A tolerance of +/- 10mm is applied to dimensions*


*Comes with a standard black cover and Handi pump*

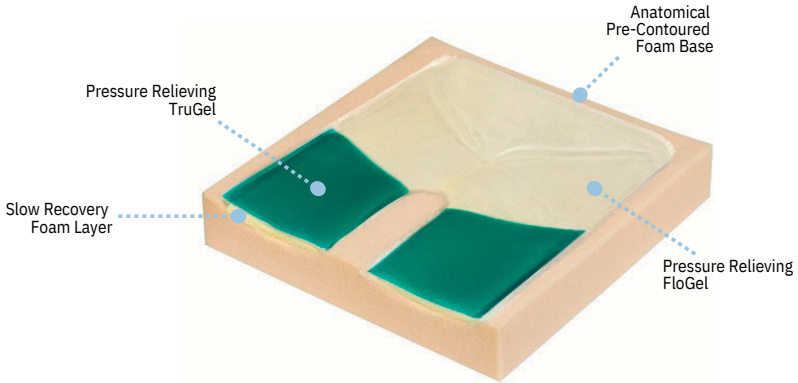

## Features & Benefits

- The Duogel combines a pre-contoured base foam with memory foam, topped with TruGel and FloGel.
- The FloGel is a soft viscous material that allows the Ischial Tuberosities to immerse reducing pressure and enhancing comfort.
- The TruGel offers stability under the legs.
- The clear film layer is easy to wipe clean.

| Product Code | Dimensions* (W x L x H)            | Cushion Weight |
|--------------|------------------------------------|----------------|
| RD003-1414   | 36cm x 36cm x 8cm / 14" x 14" x 3" | 1.2kg/2.7lb    |
| RD003-1514   | 38cm x 36cm x 8cm/ 15" x 14" x 3"  | 1.3kg/2.9lb    |
| RD003-1517   | 38cm x 43cm x 8cm / 15" x 17" x 3" | 1.8kg/4.0lb    |
| RD003-1616   | 40cm x 40cm x 8cm/ 16" x 16" x 3"  | 1.8kg/4.0lb    |
| RD003-1618   | 40cm x 45cm x 8cm/ 16" x 18" x 3"  | 1.9kg/4.2lb    |
| RD003-1717   | 43cm x 43cm x 8cm/ 17" x 17" x 3"  | 1.9kg/4.2lb    |
| RD003-1719   | 43cm x 48cm x 8cm / 17" x 19" x 3" | 2.0kg / 4.4lb  |
| RD003-1816   | 45cm x 40cm x 8cm/ 18" x 16" x 3"  | 1.9kg/4.2lb    |
| RD003-1818   | 45cm x 45cm x 8cm/ 18" x 18" x 3"  | 2.0kg/4.4lb    |
| RD003-1820   | 45cm x 50cm x 8cm/ 18" x 20" x 3"  | 2.1kg/4.6lb    |
| RD003-2018   | 50cm x 45 cm x 8cm/ 20" x 18" x 3" | 2.1kg/4.6lb    |
| RD003-2020   | 50cm x 50cm x 8cm/ 20" x 20" x 3"  | 2.5kg/5.5lb    |
| RD003-2218   | 55cm x 45cm x 8cm/ 22" x 18" x 3"  | 2.8kg/6.2lb    |
| RD003-2418   | 60cm x 45cm x 8cm/ 24" x 18" x 3"  | 3kg/6.6lb      |

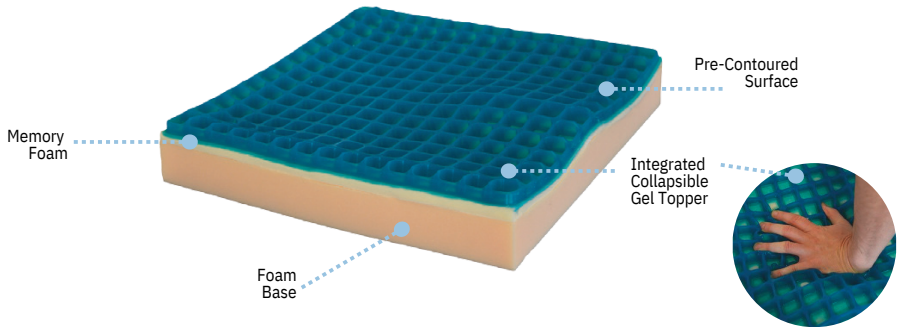

### Features & Benefits

- The DuoProfile is a 3 layer design featuring a foam base, a memory foam layer and a collapsible gel topper.
- The pre-contoured cushion assists in hip and leg alignment.
- This design allows the user to immerse into the cushion, the gel topper conforms to your shape.
- The clear film layer is easy to wipe clean.

| Product Code | Dimensions* (W x L x H)            | Cushion Weight |
|--------------|------------------------------------|----------------|
| RDP003-1616  | 40cm x 40cm x 8cm / 16" x 16" x 3" | 1.8kg/4.0lb    |
| RDP003-1717  | 43cm x 43cm x 8cm / 17" x 17" x 3" | 1.9kg/4.2lb    |
| RDP003-1816  | 45cm x 40cm x 8cm / 18" x 16" x 3" | 1.9kg/4.2lb    |
| RDP003-1818  | 45cm x 45cm x 8cm / 18" x 18" x 3" | 2kg/4.4lb      |
| RDP003-2018  | 50cm x 45cm x 8cm / 20" x 18" x 3" | 2.2kg/4.8lb    |

The weight limit for the DuoProfile cushion is 130kg/287lbs

\*A tolerance of +/- 10mm is applied to dimensions

Comes with a standard black cover. Wipeable cover available on request.



## Features & Benefits

- The Easy features 2 layers of memory foam with FloGel in the centre.
- The cushion facilitates effective pressure redistribution and immersion.
- The cushion is fully rotational, it has no top, bottom, front or back.
- This is the ideal cushion for use in the home environment.
- The clear film layer is easy to wipe clean.
- Easy is lightweight and easy to transport.

| Product Code | Dimensions* (W x L x H)            | Cushion Weight |
|--------------|------------------------------------|----------------|
| RE005-1416   | 36cm x 40cm x 5cm / 14" x 16" x 2" | 1.7kg/3.8lb    |
| RE005-1418   | 36cm x 45cm x 5cm / 14" x 18" x 2" | 1.8kg/4.0lb    |
| RE005-1616   | 40cm x 40cm x 5cm / 16" x 16" x 2" | 1.8kg/4.0lb    |
| RE005-1717   | 43cm x 43cm x 5cm / 17" x 17" x 2" | 1.9kg/4.2lb    |
| RE005-1816   | 45cm x 40cm x 5cm / 18" x 16" x 2" | 1.9kg/4.2lb    |
| RE005-1818   | 45cm x 45cm x 5cm / 18" x 18" x 2" | 2.0kg/4.4lb    |
| RE005-1820   | 45cm x 50cm x 5cm / 18" x 20" x 2" | 2.1kg/4.6lb    |
| RE005-2020   | 50cm x 50cm x 5cm / 20" x 20" x 2" | 2.2kg/4.8lb    |
| RE005-2218   | 55cm x 45cm x 5cm / 22" x 18" x 2" | 2.3kg/5.0lb    |


The weight limit for the Easy cushion is 130kg/287lbs

\*A tolerance of +/- 10mm is applied to dimensions

Comes with a wipeable black cover



Memory Foam on Both Sides Reduces 'Hammock Effect'

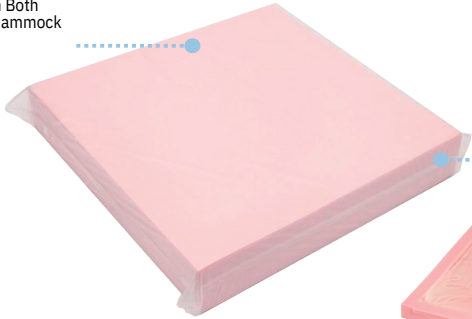

Added Height For Extra Comfort Additional Immersion Supports 190kg



FloGel Adds Comfort And Pressure Relief

## Features & Benefits


- The Easy High Profile features 2 deep layers of memory foam with FloGel in the centre.
- The cushion facilitates effective pressure redistribution and immersion.
- The additional height makes this a suitable cushion for a higher weight user.
- The cushion is fully rotational, it has no top, bottom, front or back.
- This is the ideal cushion for use in the home environment.
- The clear film layer is easy to wipe clean.

| Product Code | Dimensions* (W x L x H)                | Cushion Weight |
|--------------|--|----------------|
| REH005-1616  | 40cm x 40cm x 8.5cm / 16" x 16" x 3.3" | 1.8kg/4.0lb    |
| REH005-1717  | 43cm x 43cm x 8.5cm / 17" x 17" x 3.3" | 2.0kg/4.4lb    |
| REH005-1816  | 45cm x 40cm x 8.5cm / 18" x 16" x 3.3" | 2.1kg/4.6lb    |
| REH005-1818  | 45cm x 45cm x 8.5cm / 18" x 18" x 3.3" | 2.5kg/5.5lb    |
| REH005-1820  | 45cm x 50cm x 8.5cm / 18" x 20" x 3.3" | 2.6kg/5.7lb    |
| REH005-2020  | 50cm x 50cm x 5cm / 20" x 20" x 3.3"   | 3.0kg/6.6lb    |

The weight limit for the Easy High Profile cushion is 190kg/418lbs

\*A tolerance of +/- 10mm is applied to dimensions

Comes with a wipeable black cover

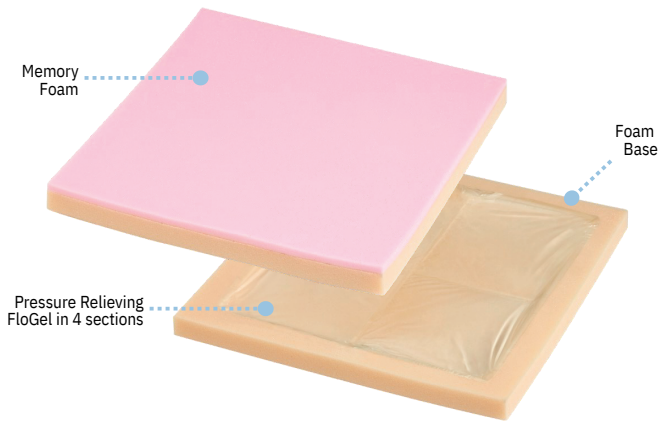

## Features & Benefits


- This foam and FloGel cushion is low profile.
- This design combines soft memory foam top layer with a FloGel centre and a foam base for comfort and pressure relief.
- The clear film layer is easy to wipe clean.
- Lightweight and easy to transport.

| Product Code | Dimensions* (W x L x H)              | Cushion Weight |
|--------------|--------------------------------------|----------------|
| RT001-1416   | 36cm x 40cm x 4cm / 14" x 16" x 1.5" | 1.2kg/2.7lb    |
| RT001-1616   | 40cm x 40cm x 4cm/ 16" x 16" x 1.5"  | 1.3kg/2.9lb    |
| RT001-1717   | 43cm x 43cm x 4cm/ 17" x 17" x 1.5"  | 1.4kg/3.1lb    |
| RT001-1816   | 45cm x 40cm x 4cm/ 18" x 16" x 1.5"  | 1.5kg/3.3lb    |
| RT001-1818   | 45cm x 45cm x 4cm/ 18" x 18" x 1.5"  | 1.6kg/3.5lb    |

*The weight limit for the Trio cushion is 80kg/176lbs*

*\*A tolerance of +/- 5mm is applied to dimensions*


*Comes with a standard black cover*

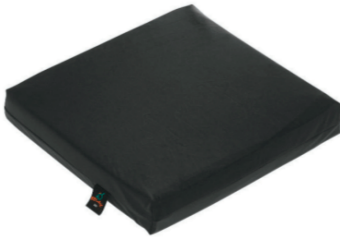

**Wipeable Black Outer Cover**

Available with:  
Easy, Easy High Profile  
(Also available for Duogel and DuoProfile on request)

Materials:  
Topside and Underside: Polyurethane transfer coating on a weft knitted polyester fabric.  
Features:  
Water resistant wipe down cover  
Easy to clean & maintain

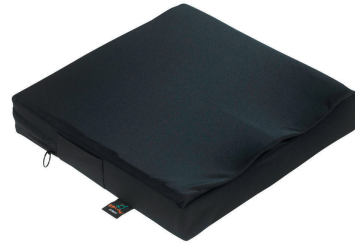

**Standard Black Outer Cover**

Available with:  
Duogel, Gelcell, Gelcell G2, DuoProfile, Trio

Materials:  
Topside: Weft knitted polyester fabric with a polyurethane transfer coating on inner side.  
Underside: Anti-slip nylon coated with flame retardant PU, which has been treated with a biocide.  
Features:  
Water resistant and easy to wash  
High Grip /Anti-slip surface

**Codes & Sizes**

| Duogel      | DuoProfile  | Easy       | Easy High Profile |
|-------------|-------------|------------|-------------------|
| DWC003-1414 | DWC003-1616 | EC004-1416 | ECH005-1616       |
| DWC003-1514 | DWC003-1717 | EC004-1418 | ECH005-1717       |
| DWC003-1517 | DWC003-1816 | EC004-1616 | ECH005-1816       |
| DWC003-1616 | DWC003-1818 | EC004-1717 | ECH005-1818       |
| DWC003-1618 | DWC003-2018 | EC004-1816 | ECH005-1820       |
| DWC003-1717 |             | EC004-1818 | ECH005-2020       |
| DWC003-1719 |             | EC004-1820 |                   |
| DWC003-1816 |             | EC004-2020 |                   |
| DWC003-1818 |             | EC004-2218 |                   |
| DWC003-1820 |             |            |                   |
| DWC003-2018 |             |            |                   |
| DWC003-2020 |             |            |                   |
| DWC003-2218 |             |            |                   |
| DWC003-2418 |             |            |                   |

**Codes & Sizes**

| Duogel     | DuoProfile | Gelcell/G2        | Trio       |
|------------|------------|-------------------|------------|
| DC003-1414 | DC003-1616 | <b>Gelcell G2</b> | TC001-1416 |
| DC003-1514 | DC003-1717 | GC005-1414        | TC001-1616 |
| DC003-1517 | DC003-1816 | GC005-1515        | TC001-1717 |
| DC003-1616 | DC003-1818 | GC005-1616        | TC001-1816 |
| DC003-1618 | DC003-2018 | GC005-1618        | TC001-1818 |
| DC003-1717 |            | GC005-1717        |            |
| DC003-1719 |            | GC005-1816        |            |
| DC003-1816 |            | GC005-1818        |            |
| DC003-1818 |            | <b>Gelcell</b>    |            |
| DC003-1820 |            | GC004-1514        |            |
| DC003-2018 |            | GC004-1616        |            |
| DC003-2020 |            | GC004-1717        |            |
| DC003-2218 |            | GC004-1816        |            |
| DC003-2418 |            | GC004-1818        |            |
|            |            | GC004-2018        |            |

**Handi Pump**

Dual action Handi Pump  
Available with:  
Gelcell, Gelcell G2



Code: AP001



[www.trulife.com](http://www.trulife.com)