


ETHNOCARE™

OVERLAY™

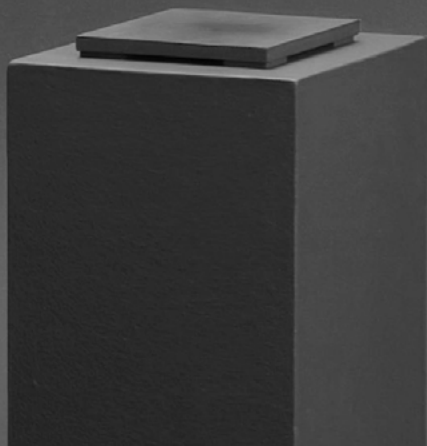

01 Redefine people's mobility

Ethnocare aims to empower individuals to achieve their full potential without limitations. The company is challenging the orthopedic industry by providing innovative solutions that improve users' lives.

In this sense, the company is putting forward the Overlay. Inspired by sportswear technology, this breakthrough transtibial prosthetic device, uses air to manage the changes in limb volume.



Problem addressed



Dimensional variation of the residual limb.



Pressure points in the socket



Misalignment of the limb in the socket




Our solution The Overlay

The Overlay is a breathable fabric sleeve with an integrated air expansion system that the wearer can adjust for maximum, personalized comfort.

OVERLAY™



Personalized fit

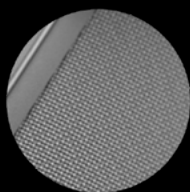

Retrofit sizing



Air expansion system



watch the video

02 Overlay's technical details

Personalized fit



The Overlay is a stretchy sports textile sleeve featuring a unique air expansion system for a **customized, comfortable and uniform fit** in the socket.

Air expansion system



The Overlay incorporates an **air expansion system** as a replacement for prosthetic stockings, enabling more effective residual limb volume management.

Retrofit sizing



The Overlay incorporates **advanced fabric technology** with a total thickness of just under one millimeter. It's designed to be compatible with most liners and sockets, allowing you to continue using your existing equipment without disruption.

Suspension method



The Overlay can easily be used with the **majority of existing liners** and is **compatible** with distal fixation, suspension sleeves, and passive vacuum systems. It provides amputees with increased **adjustability, mobility,** and freedom to live their best life.

03

How to use



①

Place the liner on the residual limb



②

Slide the Overlay over the liner with the air expansion system behind the limb



③

The prosthesis can then be slid back on. All the air cells must be in the socket below the trim line.



④

Use the pump to adjust the cushioning as you wish throughout the day.



⑤

You can also use the integrated pump



Now you are in control !



Simply deflate the Overlay before removing the prosthesis and the liner.

04

Added values

The Ethnocare Overlay is a simple and intuitive prosthetic device with adjustable cushioning from 2 to 15 ply.

It can be used through clothing without removing the prosthesis and evenly distributes air within the socket to adapt to different residual limb morphologies.

**Get in contact
with Ethnocare for
more information**





Contact us



Ethnocare.ca



Admin@ethnocare.ca



[https://www.linkedin.com/
company/ethnocare/](https://www.linkedin.com/company/ethnocare/)



@ethnocare_canada



ethnocare.ca