


To determine the most suitable size for the patient, the measurement protocol involves identifying the suspension type, measuring the circumference and length at a specific height, and selecting the appropriate product orientation.


STEP 1

which circumference ?

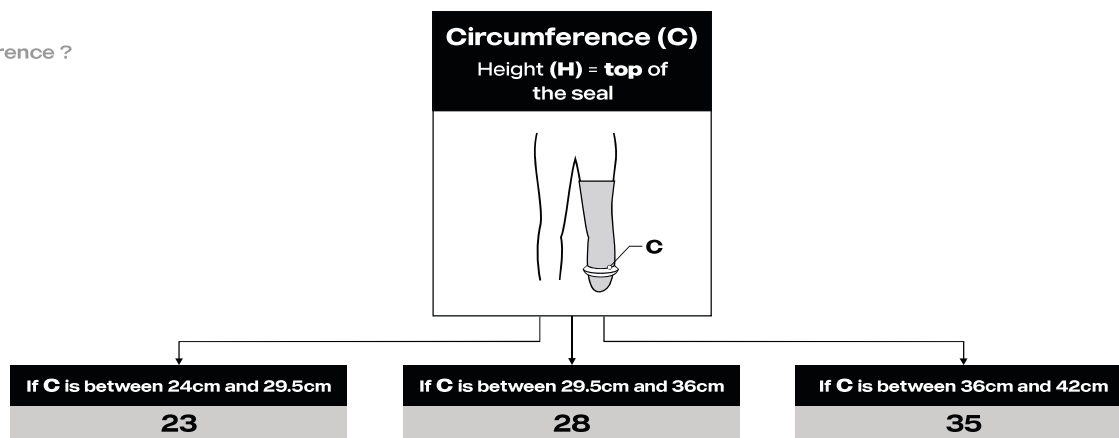

STEP 2


which length ?

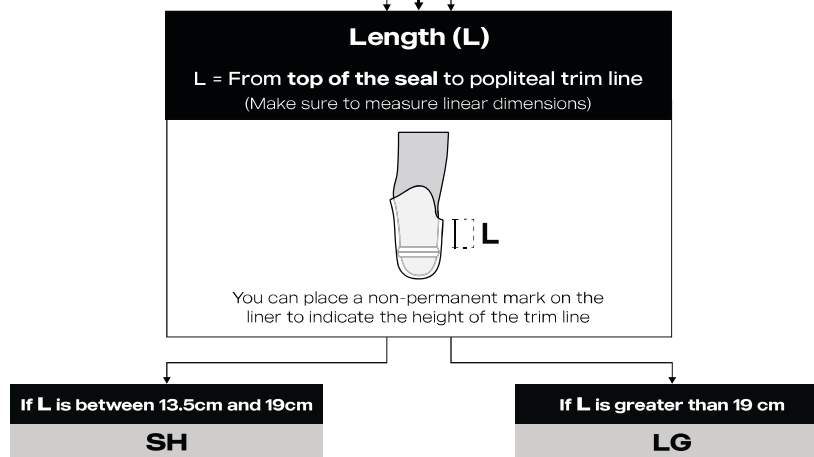

If patient do not have a socket yet


Length (L)

L = From top of seal to popliteal fossa
(Make sure to measure linear dimensions)

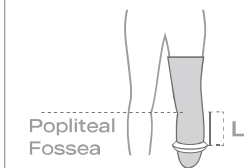


STEP 3

which side ?

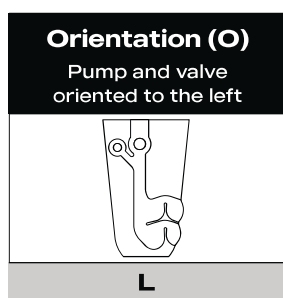



Orientation is based on user preference

Most users choose the same orientation as the side where the Overlay will be worn

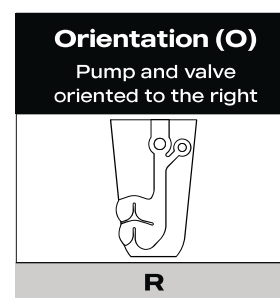

Suspension	Orientation	Length	Circumference	Code
Distal vacuum	Right	13.5-19 cm	24-29.5 cm	OV-23-SH-R
			29.5-36 cm	OV-28-SH-R
			36-42 cm	OV-35-SH-R
		+19 cm	24-29.5 cm	OV-23-LG-R
			29.5-36 cm	OV-28-LG-R
			36-42 cm	OV-35-LG-R
	Left	13.5-19 cm	24-29.5 cm	OV-23-SH-L
			29.5-36 cm	OV-28-SH-L
			36-42 cm	OV-35-SH-L
		+19 cm	24-29.5 cm	OV-23-LG-L
			29.5-36 cm	OV-28-LG-L
			36-42 cm	OV-35-LG-L

Compatibility : Distally positionned seal with or without sleeves

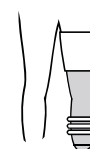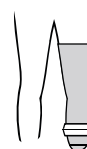

Compatible with :

Össur, Ottobock, Alps, WillowWood liners


And other similar liners

Please see next page for
Other systems →

To determine the most suitable size for the patient, the measurement protocol involves identifying the suspension type, measuring the circumference and length at a specific height, and selecting the appropriate product orientation.


STEP 1

which circumference ?

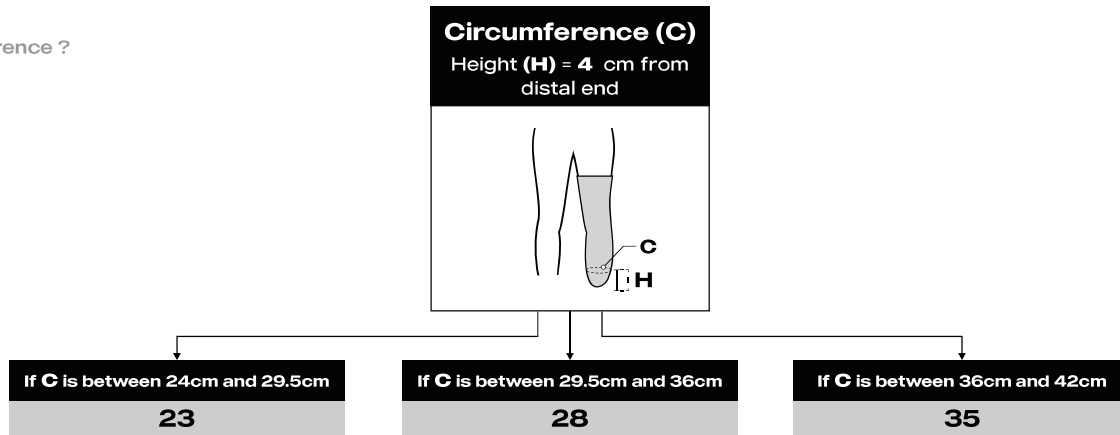

STEP 2


which length ?

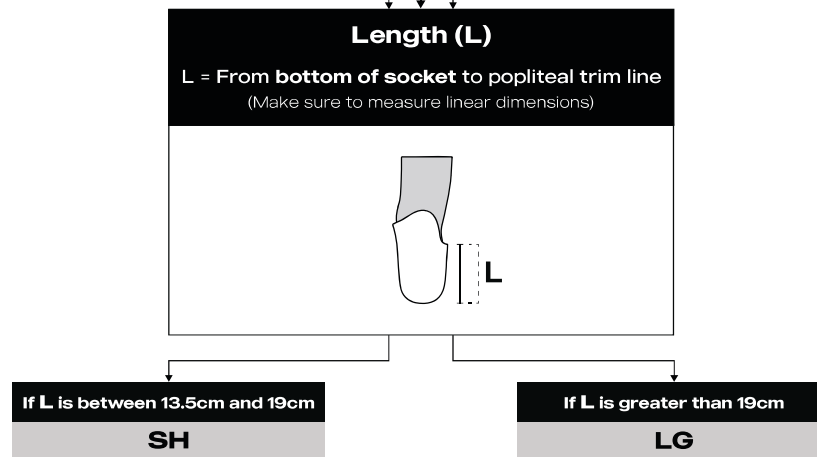

If patient do not have a socket yet


Length (L)

L = From end of liner to popliteal fossa
(Make sure to measure linear dimensions)

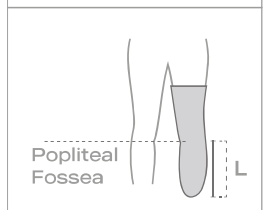


STEP 3

which side ?

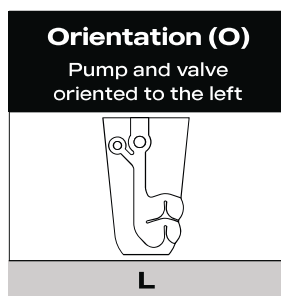


Orientation is based on user preference

Most users choose the same orientation as the side where the Overlay will be worn

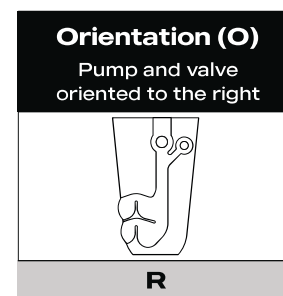

Suspension	Orientation	Length	Circumference	Code
Distal locking Suspension sleeve Cushion Lanyards	Right	13.5-19 cm	24-29.5 cm	OV-23-SH-R
			29.5-36 cm	OV-28-SH-R
			36-42 cm	OV-35-SH-R
		+19 cm	24-29.5 cm	OV-23-LG-R
			29.5-36 cm	OV-28-LG-R
			36-42 cm	OV-35-LG-R
	Left	13.5-19 cm	24-29.5 cm	OV-23-SH-L
			29.5-36 cm	OV-28-SH-L
			36-42 cm	OV-35-SH-L
		+19 cm	24-29.5 cm	OV-23-LG-L
			29.5-36 cm	OV-28-LG-L
			36-42 cm	OV-35-LG-L

Compatibility :

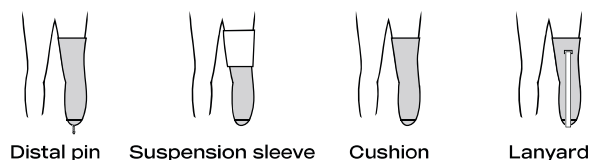

Please see next page for
Distal seal →